

Fitting Instructions

STEP 1



Lightly dust inside of the cover with talcum powder (to ensure a smooth application over the body part and avoid sticking together).

STEP 2



Fold the top of the cast cover down 2-3 rolls (to ensure the cover will not tear while applying).

STEP 3



Slide cover slowly over body part/cast to be protected. Once applied ensure the cover has a tight smooth seal (of approx 20 mm) onto the skin above the cast.

STEP 4



If using for bathing or swimming, place a finger between the seal and skin to allow air inside the cover to be released as submerging body part into the water. Remove finger before fully submerging.

Detailed fitting instructions for specific uses:

Fitting over Casts and Bandages:

1. File down any sharp or rough areas on the cast which could cause the cover to tear (particularly on fibreglass casts). Fine sandpaper or an emery board is suitable.
2. Lightly dust inside of the cover with a small quantity of talcum powder to ensure a smooth application over the body part and avoid the cover sticking together. (See photo in Step 1).
3. Fold the opening of the Dricast cover over 2-3 times to avoid possible tearing when applying. (See photo in Step 2).
4. Using the thumbs and fingers of both hands stretch the opening wide enough to allow the cover to go over the cast or bandaged body part, pull the Dricast cover up to completely cover the cast or bandage. (See photos in Steps 2 & 3).
5. If using for bathing or swimming, place a finger between the seal and skin to allow air inside the cover to be released as submerging the body part into the water. Remove finger before fully submerging. (See photo in Step 4).
6. Allow the cover to seal onto the skin above the cast with approx 20 mm of the cover sealed flat onto the skin.
7. Ensure a tight seal is maintained while using the Dricast product.

8. If the cover is too tight for the body part being used upon, contact us for instruction on how to adjust.

Fitting over Ulcers, Burns and other open wounds:

1. Ensure that the open wound is covered with a sterile wound management covering. The Dricast product is a waterproof cover – IT IS NOT A STERILE PRODUCT.
2. Lightly dust inside of the cover with a small quantity of talcum powder to ensure a smooth application over the body part and avoid the cover sticking together. (See photo in Step 1).
3. Fold the opening of the Dricast cover over 2-3 times to avoid possible tearing when applying. (See photo in Step 2).
4. Using the thumbs and fingers of both hands stretch the opening wide enough to allow the cover to go over the bandaged wound, pull the Dricast cover up to completely cover the bandaged wound. (See photos in Steps 2 & 3).
5. If using for bathing or swimming, place a finger between the seal and skin to allow air inside the cover to be released as submerging the body part into the water. Remove finger before fully submerging. (See photo in Step 4).
6. Allow the cover to seal onto the skin above the bandaged wound with approx 20 mm of the cover sealed flat onto the skin.
7. Ensure a tight seal is maintained while using the Dricast product.
8. If the cover is too tight for the body part being used upon, contact us for instruction on how to adjust.

Prosthetic Limbs:

1. Leave your prosthetic limb in place on your body.
2. Lightly dust inside of the cover with a small quantity of talcum powder to ensure a smooth application and avoid the cover sticking together. (See photo in Step 1).
3. Fold the opening of the Dricast cover over 2-3 times to avoid possible tearing when applying. (See photo in Step 2).
4. Using the thumbs and fingers of both hands stretch the opening wide enough to allow the cover to go over the prosthetic limb, pull the Dricast cover up to completely cover the prosthetic limb. (See photos in Steps 2 & 3).
5. If using for bathing or swimming, place a finger between the seal and skin to allow air inside the cover to be released as submerging into the water. Remove finger before fully submerging. (See photo in Step 4).
6. Allow the cover to seal onto the skin above the prosthetic limb with approx 20 mm of the cover sealed flat onto the skin.
7. Ensure a tight seal is maintained while using the Dricast product.
8. If the cover is too tight for the area being used upon, contact us for instruction on how to adjust.