

CARE REQUIRED FOR YOUR DRICAST COVER:

- **DO NOT WALK WITH YOUR DRICAST LEG COVER ON**
 - Your Dricast leg cover should only be worn while showering, bathing or swimming. Walking with your Dricast leg cover on could cause it to weaken and possibly leak
- **AVOID SHARP OBJECTS**
 - Avoid any sharp objects that may pierce the cover
- **15–20 MINUTE USAGE PERIOD**
 - To avoid condensation build up inside the cover, only use your Dricast cover for 15 to 20 minutes at any one time. Allow to dry naturally out of direct light for approx 2 hours between uses
- **STORAGE**
 - Store in a cool, dry place away from any direct light source
- **REHABILITATION USE**
 - If using your Dricast cover for medical rehabilitation purposes, ensure you consult your physician prior to use

CAUTION:

- The product becomes **slippery when wet**. Ensure extra care is taken when wearing leg models to avoid slipping. Use of a towel on the floor of the shower tray and tile floor is recommended
- This product contains **natural rubber latex** which may cause allergic reactions in some people
- **Keep out of reach of children** – ensure adult supervision for use with children
- Correct use of Dricast Covers should keep wounds and injuries dry in most all cases. However, 100% protection cannot be guaranteed, if medical advice requires the wound/injury does not become wet under any circumstances, consult your physician before using the product
- The product is **NOT STERILE**. It is designed for protection from water only. **DO NOT PLACE OVER AN OPEN WOUND**. Ensure any open wound is covered by a sterile covering before use
- The product has to fit tightly to the skin to ensure water does not leak in. If however circulation becomes restricted, remove cover and contact supplier for instructions on how to enlarge the opening